

**【Product Name】** Sample Preservative Fluid

**【Packing Size】** 1mL/tube, 2mL/tube, 3mL/tube, 4mL/tube

**【Usage】** For preservation of nucleic acid in the virus sample.

**【Principle and Advantage】**

This product contains ingredients that effectively inhibit DNase/RNase activity and can stably preserve viral nucleic acid in various samples for a long time at -20°C without degradation. This product is simple and convenient to use, can be widely used in hospitals, research institutes and others to collect and transport virus samples under room temperature.

**【Main components】**

Cat#	Volume	Tube	Sample
BSC82N1	1mL	2mL	Nasopharyngeal swabs
BSC82X1-A	2mL	5mL	
BSC82X1-B	3mL	5mL	
BSC82X1-C	4mL	5mL	Anal swabs

**【Storage and Shelf Life】**

1. The kit can be stored at room temperature (15-25°C) away from light.
2. The kit can be stored for 12 months stably

**【Sample Collection】**

**Nasopharyngeal swab:** Wipe bilateral tonsils and posterior pharyngeal wall with moderate intensity force. Avoid touching the tongue. After collection, place the swab into a sample storage tube immediately. Place the sample storage tube at 2-8 °C for up to one month, for long time storage, please store at -20 °C.

Before sample extraction, shake the swab storage tube vigorously for 1 min, and

take 200-300 µL of soaking solution for testing.

**Anal swab** Insert the sampling swab into the anus 2-3cm, and swab from the wrinkles around the anus, or gently rotate and wipe inside the anus. After collection, place the swab into a sample storage tube immediately. Place the sample storage tube at 2-8 °C for up to one month, for long time storage, please store at -20 °C.

Before sample extraction, shake the swab storage tube vigorously for 1 min, and take 200-300 µL of soaking solution for testing.

**【Notes】**

When taking multiple samples at the same time, take care to prevent cross-contamination between samples.

**【Company Information】**

Manufacturer: Hangzhou Bioer Technology Co.,Ltd

Address: No.1192 Bin'An Rd, Binjiang District, Hangzhou, Zhejiang Province, China

Tel: 0571-87774567 Fax: 0571-87774553

Web: [www.bioer.com.cn](http://www.bioer.com.cn)

Zip Code: 310053

Aftersales Service Provider: Hangzhou Bioer Technology Co.,Ltd